



PRODUCT GUIDE

WHY ROHO?

Demand ROHO® for the best clinical outcomes. ROHO is the pioneer of adjustable, air-filled, cellular designed wheelchair support surfaces. Through intense research and development, ROHO developed DRY FLOATATION® Technology, mimicking the pressure redistributing properties of water. DRY FLOATATION Technology was designed to conform to an individual's seated shape and recommended for individuals who may have a history of, or currently have skin breakdown.

- ✓ SUPERIOR Skin & Soft Tissue Protection
- ✓ LONGER Safe Sitting Times
- ✓ Positioning and Stability Capabilities



THE SCIENCE BEHIND ROHO

Introduction	5
Benefits of DRY FLOATATION®	6
The Four Principles	7

SEAT CUSHIONS

Smart Check	10-11
Adjustable Skin Protection	12-15
Adjustable Skin Protection & Positioning	16-21
Air/Foam Combination Cushions	22-25
Additional Options	26-29
Cushion Accessories	30-33

SPECIALTY APPLICATIONS PRODUCTS

Specialty Application Products	36-41
--	-------

WHEELCHAIR BACKS

AGILITY® Back System	44-47
AGILITY® Accessories	48-49

MATTRESS OVERLAYS

Mattress Overlays	52-55
-----------------------------	-------

RESOURCES

Cushion Sizing Chart	58-61
Wheelchair Back System Sizing Chart	62-63
Cushion Comparison Chart	64-65
Returns and Warranty Information	66
Quality Management	67

THE SCIENCE BEHIND



ROHO®

Recommend the Best Cushion – Backed by Clinical Evidence

ROHO has been validated by industry clinicians and scientists for over 40 years. With decades of experience in the market and over 90 scientific and clinical studies, ROHO is the efficacy and outcomes expert.



50

Single-Patient
Case Studies



30

Peer-Reviewed
Scientific Journal Articles



20

Hospital & Home-Care
Based Studies



ZERO COMPETITION with this Magnitude of Evidence-Based Research!

With over 1 million ROHO products in use in over 65 countries worldwide, you can be confident in using ROHO products.

ROHO products are unique in their capability to conform and adapt to the body's shape. No other product has DRY FLOATATION® Technology to evenly distribute the forces of body weight and reduce the effects of friction and shear on tissue. DRY FLOATATION® Technology creates the ideal healing environment by minimizing soft tissue deformation while maintaining blood flow.

Time and time again, studies have proven that ROHO is the optimal choice for preventing and treating pressure ulcers and post-operative wounds.

Quality and efficacy are the key foundations of ROHO. To ensure that ROHO products continue to achieve the best clinical outcomes, ROHO has dedicated extensive resources into building industry-leading clinical research for ROHO products. To continuously improve and stay above competition, ROHO has formed a Scientific Advisory Board composed of an esteemed group of subject matter experts. Their efforts and research drive the company from research and development through training and education.

ROHO[®] Cushions:

10,000x

more effective than foam in reducing tissue damage risk.*

41%

decrease in tissue stress from the intimate way they conform to the body.*

57%

better immersion than foam to protect the tissues from further risk posed by disuse-related changes, including muscle atrophy, bone shape adaptations, spasms, obesity, and scarring.*


*Gefen, et al, (JTV 2014)

Discover the Four Principles of DRY FLOATATION® Technology

Low Surface Tension

Definition: The ability of the surface to easily yield upon contact.

ROHO's unique cellular design and neoprene material properties allow the body to maximally penetrate the cushion with minimal soft tissue deformation.

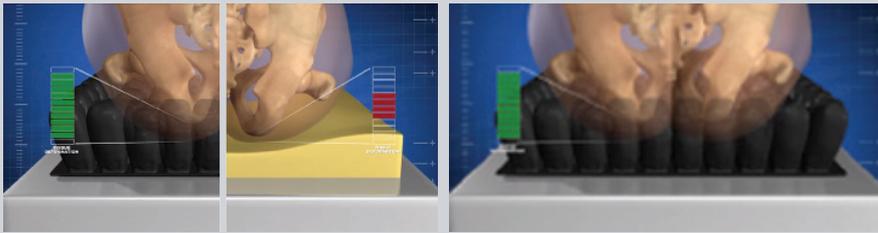

Constant Restoring Forces

Definition: Fluid flotation provides even and constant forces at all points of contact.

ROHO's unique cellular design, neoprene material properties and controlled airflow between cells provide flotation, which minimizes soft tissue deformation regardless of change in position.



Six Degrees of Freedom

Definition: The freedom of movement in all directions.

ROHO's unique cellular design and neoprene material properties compress, bend and twist to minimize shear, friction and soft tissue deformation of the body, especially with movement.



Low Friction and Shear

Definitions: Friction is the resistance of movement between two surfaces. Shear is the result of materials or tissues being pulled in opposite directions.

ROHO's unique cellular design and neoprene material properties, in combination with the first three principles, allow for a dramatic reduction of soft tissue deformation from external friction and both external and internal shear forces.







SEAT CUSHIONS

SEAT CUSHIONS

Smart Check[®]



Smart Check[®] offers real-time feedback to monitor the cushion's set inflation range with the push of a button.

Clinicians can confidently recommend ROHO with Smart Check knowing their clients can easily check and maintain their ROHO cushion. Smart Check helps individuals achieve maximum skin and soft tissue protection while offering instant feedback and peace of mind. 88% of Smart Check users report that they are more confident using their ROHO cushion with Smart Check.

As the world's best personal electronic cushion security device, Smart Check provides individuals with the best cushion technology. Smart Check is designed to work exclusively with ROHO[®] Sensor Ready[™] HIGH PROFILE[®] and MID PROFILE[™] cushions.



ROHO® HIGH PROFILE® Sensor Ready™ Cushion

The HIGH PROFILE Single Compartment Cushion with Sensor Ready Technology is an adjustable, air-filled, cellular-designed cushion. The cushion provides a wheelchair support surface that utilizes DRY FLOATATION® Technology intended to conform to an individual's seated shape. The cushion provides skin and soft tissue protection in an environment to facilitate wound healing and prevention.

Sensor Ready Cushions are intended for use with Smart Check® by ROHO®, which is a cushion feedback system that is intended to assist an individual by saving an inflation range during cushion setup. During subsequent inflation level checks, Smart Check is intended to indicate whether adjustments are needed to remain in the saved range.

DIMENSIONS

Width: 11.5" to 26.75" / 29 cm to 68 cm
Height: 4.25" / 10.5 cm
Weight: Approx. 3.25 lbs. / 1.5 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion & Smart Check:
24 Months
Cover: 6 Months

■ SKIN PROTECTION □ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT



ROHO® MID PROFILE™ Sensor Ready™ Cushion

The MID PROFILE Single Compartment Cushion with Sensor Ready Technology has a lower cell height than the HIGH PROFILE™ Cushion, providing an individual with skin and soft tissue protection.

Sensor Ready Cushions are intended for use with Smart Check® by ROHO®, which is a cushion feedback system that is intended to assist an individual by saving an inflation range during cushion setup. During subsequent inflation level checks, Smart Check is intended to indicate whether adjustments are needed to remain in the saved range.

DIMENSIONS

Width: 13.25" to 20" / 33.5 cm to 50.5 cm
Height: 3.25" / 8.5 cm
Weight: 3.0 lbs. / 1.4 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion & Smart Check:
24 Months
Cover: 6 Months

■ SKIN PROTECTION □ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT

SEAT CUSHIONS

ADJUSTABLE SKIN PROTECTION



ROHO® HIGH PROFILE®

Height: Approximately 4.25" / 10.5 cm
Immersion: Approximately 3.75" / 9 cm



ROHO® MID PROFILE™

Height: Approximately 3.25" / 8.5 cm
Immersion: Approximately 2.75" / 7 cm



ROHO® LOW PROFILE®

Height: Approximately 2.25" / 5.5 cm
Immersion: Approximately 1.75" / 4 cm

ROHO's single valve cushions are the original DRY FLOATATION® Technology cushion, invented specifically for skin protection.

For over 40 years, ROHO's cushions have made an impact on millions of people's lives by offering an optimal environment for individuals with skin and soft tissue integrity issues. The cellular design and the soft pliable material immerse the individual into the cushion for maximum pressure redistribution, while decreasing friction and shear. By providing the individual with an air-filled, cellular surface the cushion adjusts to the shape of the individual, minimizing tissue deformation and maintaining blood flow.

ROHO® HIGH PROFILE® Single Compartment Cushion

The HIGH PROFILE Single Compartment Cushion is made of 4.25" / 10.5 cm tall interconnected air cells that provide an adjustable, low-deformation environment that assists in the healing and prevention of pressure ulcers. The HIGH PROFILE Cushion provides a greater contact area with the opportunity for maximum immersion and envelopment to manage the individual's load.

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk of skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown and/or any stage pressure ulcer (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 3 inches / 7.5 centimeters



DIMENSIONS

Width: 11.5" to 26.75" / 29 cm to 68 cm

Height: 4.25" / 10.5 cm

Weight: Approx. 3.25 lbs. / 1.5 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 24 Months

Cover: 6 Months

■ SKIN PROTECTION □ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT



DIMENSIONS

Width: 13.25" to 20" / 33.5 cm to 50.5 cm

Height: 3.25" / 8.5 cm

Weight: Approx. 3.0 lbs. / 1.4 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 24 Months

Cover: 6 Months

■ SKIN PROTECTION □ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT

ROHO® MID PROFILE™ Single Compartment Cushion

The MID PROFILE Single Compartment Cushion is made of 3.25" / 8.5 cm tall interconnected air cells that provide an optimal seating environment to facilitate wound healing. Made with ROHO's DRY FLOATATION® Technology, the cushion is intended to conform to an individual's seated shape, minimizing tissue deformation and maintaining blood flow. The MID PROFILE™ Cushion provides a greater contact area with the opportunity for moderate immersion and envelopment to manage the individual's load.

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown and/or any stage pressure ulcer (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 2.25 inches / 5.5 centimeters

ROHO® LOW PROFILE® Single Compartment Cushion

The LOW PROFILE Single Compartment Cushion is made of 2.25" / 5.5 cm tall interconnected air cells that maximize mobility (transfers) for the individual while protecting them from a potential risk of tissue breakdown. The LOW PROFILE Cushion provides immersion and envelopment to manage the individual's load.

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 1 inch / 2.5 centimeters



DIMENSIONS

Width: 11.5" to 23.5" / 29 cm to 59.5 cm

Height: 2.25" / 5.5 cm

Weight: Approx. 2.5 lbs. / 1.1 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 24 Months

Cover: 6 Months

■ SKIN PROTECTION □ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT

SEAT CUSHIONS

ADJUSTABLE SKIN PROTECTION & POSITIONING



ROHO's ISOFLO® Memory Control, located at the front of all Select Series cushions, allows air to flow through the cushion in two ways:

- With the ISOFLO Memory Control open, air can flow freely throughout the entire cushion, similar to a ROHO Single Compartment Cushion.
- With the ISOFLO Memory Control closed, air becomes locked into four separate quadrants offering positioning capabilities.

ROHO sets the standard for performance in the wheelchair seating industry.

Based on over 40 years of research and individual experience, ROHO® DRY FLOATATION® Technology provides unparalleled performance in facilitating blood flow by fitting and matching the shape of an individual.

Select Series cushions are one of ROHO's most popular product lines because of the revolutionary ISOFLO Memory Control, which offers shape fitting capabilities with the push of a knob. The ISOFLO Memory Control offers a seated individual an adjustable, quick and simple positioning and stability solution.

Additional adjustable skin protection and positioning options, including a series of cushions available with dual valves, provide individuals with increased positioning and stability functions based on their needs.

ROHO® QUADTRO SELECT® HIGH PROFILE® Cushion

The QUADTRO SELECT HIGH PROFILE Cushion provides maximum skin and soft tissue protection. With the ISOFLO® Memory Control function, the individual has the ability to easily set and save positioning settings by regulating air in four quadrants. The revolutionary ISOFLO Memory Control makes it easy to adjust the individual's posture to the desired sitting position. By pushing the knob to the closed position, the user can maintain the desired sitting posture for maximum function.

The QUADTRO SELECT HIGH PROFILE Cushion is 4.25" / 10.5 cm tall, providing the maximum immersion for patients.

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, and/or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 3 inches / 7.5 centimeters
- Require increased sitting stability
- Require correction of pelvic asymmetry (medial lateral and/or anterior posterior)
- Require different positioning of the left and right femur



DIMENSIONS

Width: 12.75" to 24.5" / 32.5 cm to 62.5 cm

Height: 4.25" / 10.5 cm

Weight: Approx. 3.5 lbs. / 1.6 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 36 Months

Cover: 6 Months

■ SKIN PROTECTION ■ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT



DIMENSIONS

Width: 14.5" to 19.5" / 36.5 cm to 49.5 cm

Height: 3.25" / 8.5 cm

Weight: Approx. 3.0 lbs. / 1.3 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 36 Months

Cover: 6 Months

■ SKIN PROTECTION ■ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT

ROHO® QUADTRO SELECT® MID PROFILE™ Cushion

Similar to the HIGH PROFILE cushion, the QUADTRO SELECT MID PROFILE cushion provides maximum skin and soft tissue protection with the ability to easily set and save positioning settings. The MID PROFILE cushion has a smaller cell height than the HIGH PROFILE, at 3.25" / 8.5 cm tall. The MID PROFILE cushion offers the ISOFLO® Memory Control to adjust the user's posture to the desired sitting position.

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, and/or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown and/or any stage pressure ulcer (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, approximately up to 2 inches / 5 centimeters
- Require increased sitting stability
- Require correction of pelvic asymmetry (medial lateral and/or anterior posterior)
- Require different positioning of the left and right femur

ROHO® QUADTRO SELECT® LOW PROFILE® Cushion

The QUADTRO SELECT LOW PROFILE Cushion is the lowest profile of all the Select Series cushions with a cell height of 2.25" / 5.5 cm tall. The LOW PROFILE cushion offers ROHO's DRY FLOATATION® Technology providing maximum skin and soft tissue protection, while offering the benefits of the ISOFLO® Memory Control.

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, and/or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, approximately up to 1 inch / 2.5 centimeters
- Require increased sitting stability
- Require correction of pelvic asymmetry (medial lateral and/or anterior posterior)
- Require different positioning of the left and right femur



DIMENSIONS

Width: 11" to 24.5" / 28 cm to 62.5 cm

Height: 2.25" / 5.5 cm

Weight: Approx. 2.25 lbs. / 1 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 36 Months

Cover: 6 Months

■ SKIN PROTECTION ■ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT



ROHO® CONTOUR SELECT® Cushion

The CONTOUR SELECT cushion stabilizes the pelvis and centers the individual comfortably in the middle of the cushion with the unique, multiple cell height design. The ISOFLO® Memory Control locks air into four separate quadrants, providing stability and positioning by minimizing side-to-side and front-to-back motion.

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, and/or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown and/or any stage pressure ulcer (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 1 in. / 2.5 cm
- Require increased sitting stability, lower extremity alignment, and/or correction of pelvic asymmetry (medial lateral AND anterior posterior)
- Require different positioning of the left and right femur

DIMENSIONS

Width: 16" to 24.5" / 41 cm to 62.5 cm
 Height: 2.25" to 4.25" / 5.5 cm to 10.5 cm
 Weight: Approx. 3.0 lbs. / 1.4 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 36 Months
 Cover: 6 Months

■ SKIN PROTECTION ■ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT



ROHO® ENHANCER® Cushion

The ENHANCER Cushion uses positioning with a multiple cell height, two-compartment system for enhanced midline channeling of the femurs, lateral stability and skin/soft tissue protection.

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, and/or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown and/or any stage pressure ulcer (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 1 in. / 2.5 cm
- Require increased sitting stability and/or lower extremity alignment
- Require correction of pelvic asymmetry (medial lateral)

DIMENSIONS

Width: 14.75" to 20" / 37.5 cm to 50.5 cm
 Height: 2.25" to 4.25" / 5.5 cm to 10.5 cm
 Weight: Approx. 3.0 lbs. / 1.4 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 24 Months
 Cover: 6 Months

■ SKIN PROTECTION ■ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT



ROHO® HIGH PROFILE® Dual Compartment Cushion

The HIGH PROFILE Dual Compartment cushion is 4.25" / 10.5 cm tall and each compartment can be adjusted independently to increase positioning and stability for either side-to-side (to adjust for pelvic obliquities) or front-to-back control (to adjust bucket or tilt angle).

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, and/or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown and/or any stage pressure ulcer (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 3 in. / 7.5 cm
- Require increased sitting stability
- Require correction of pelvic asymmetry (medial lateral and/or anterior posterior)

DIMENSIONS

Width: 14.75" to 20" / 37.5 cm to 50.5 cm
Height: 4.25" / 10.5 cm
Weight: Approx. 3.25 lbs. / 1.5 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 24 Months
Cover: 6 Months

■ SKIN PROTECTION ■ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT



ROHO® LOW PROFILE® Dual Compartment Cushion

THE LOW PROFILE Dual Compartment cushion is 2.25" / 5.5 cm tall and each compartment can be adjusted independently to increase positioning and stability for either side-to-side (to adjust for pelvic obliquities) or front-to-back control (to adjust bucket or tilt angle).

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, and/or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown and/or any stage pressure ulcer (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 1 in. / 2.5 cm
- Require increased sitting stability
- Require correction of pelvic asymmetry (medial lateral or anterior posterior)

DIMENSIONS

Width: 14.75" to 20" / 37.5 cm to 50.5 cm
Height: 2.25" / 5.5 cm
Weight: Approx. 2.5 lbs. / 1.1 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 24 Months
Cover: 6 Months

■ SKIN PROTECTION ■ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT

SEAT CUSHIONS

AIR/FOAM COMBINATION CUSHIONS



ROHO has the perfect hybrid solution.

ROHO combines the stability of a contoured foam base with the skin and tissue protection for which ROHO DRY FLOATATION® cushions are known. The contoured foam base is designed to allow for increased stability for transfers and provides exceptional comfort, positioning and stability.

ROHO® Hybrid Elite® Single Compartment Cushion

The Hybrid Elite Single Compartment Cushion is an adjustable, air-filled, wheelchair support surface combining a ROHO DRY FLOATATION® cellular design cushion insert with a contoured foam base. This cushion is intended to conform to an individual's seated shape in the pelvic loading area.

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, and/or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown and/or any stage pressure ulcer (including deep tissue injury)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 2 inches / 5 centimeters
- Require increased stability
- Require lower extremity alignment
- Require more stable environment for transfers



*Cover is required for use.

DIMENSIONS

Width: 13.75" to 23.75" / 35 cm to 60.5 cm
Height: 4.25" / 11 cm
Weight: Approx. 3 lbs. / 1.4 kg

WEIGHT LIMIT

500 lbs. / 225 kg

LIMITED WARRANTY

Cushion: 24 Months

■ SKIN PROTECTION ■ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT



*Cover is required for use.

ROHO® Hybrid Elite® Dual Compartment Cushion

Similar to the Hybrid Elite Single Compartment Cushion, the Hybrid Elite Dual Compartment Cushion is an adjustable, air-filled, wheelchair support surface combining a ROHO DRY FLOATATION® cellular design cushion insert with a contoured foam base. The dual valve feature enhances the positioning element of this DRY FLOATATION insert.

DIMENSIONS

Width: 13.75" to 19.75" / 35 cm to 60.5 cm
 Height: 4.25" / 11 cm
 Weight: Approx. 3 lbs. / 1.4 kg

WEIGHT LIMIT

500 lbs. / 225 kg

LIMITED WARRANTY

Cushion: 24 Months

- SKIN PROTECTION
- POSITIONING
- ADJUSTABILITY
- ADAPTABILITY
- COMFORT



*Cover is required for use.

ROHO® nexus SPIRIT® Cushion

The nexus SPIRIT Cushion combines stability of a contoured foam base with the skin and soft tissue of a ROHO DRY FLOATATION® Cushion. The contoured foam base is designed to allow for increased stability for transfers and provides positioning of the pelvis and lower extremities for enhanced sitting posture.

Based on clinical, scientific or engineering evidence, this product may be suitable for individuals who:

- Have a history of or are at risk for skin/soft tissue breakdown and/or any category/stage pressure ulcer (including deep tissue injury), skin grafts, and/or myocutaneous flap surgery
- Currently have skin/soft tissue breakdown and/or any stage pressure ulcer (including deep tissue injury)
Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 1 in. / 2.5 cm
- Require increased sitting stability and/or lower extremity alignment
- Require a more stable environment for transfers.

DIMENSIONS

Width: 14" to 20" / 35.5 cm to 51 cm
 Height: 3.5" / 9 cm
 Weight: Approx. 3 lbs. / 1.3 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 24 Months

- SKIN PROTECTION
- POSITIONING
- ADJUSTABILITY
- ADAPTABILITY
- COMFORT



*Cover is required for use.

ROHO® HARMONY® Cushion

The HARMONY Cushion is a wheelchair support surface with a soft contoured foam base, combined with an air-filled, segmented-design overlay, intended to conform to an individual's seated shape to provide skin/soft tissue protection and positioning. The HARMONY Cushion is composed of a lightweight and comfortable ROHO AIR FLOATATION™ DEHP-free polyvinylchloride cushion with a contoured foam base.

DIMENSIONS

Width: 13.75" to 20" / 35 cm to 51 cm
 Height: 2.5" to 4.5" / 6.5 cm to 11.5 cm
 Weight: Approx. 1.25 lbs. / 0.6 kg

WEIGHT LIMIT

275 lbs. / 125 kg

LIMITED WARRANTY

Cushion: 18 Months

■ SKIN PROTECTION ■ POSITIONING ■ ADJUSTABILITY ■ ADAPTABILITY ■ COMFORT

Can't find what you're looking for?

Visit ROHO.com to:

- Locate a distributor
- Find more product information
- Learn more about the science and technology behind ROHO products
- Register a ROHO product
- Read frequently asked questions
- Watch how-to videos
- Purchase ROHO accessories

Contact your ROHO Distributor or contact ROHO Customer Care at cc@roho.com, or call (618) 277-9150.

SEAT CUSHIONS

ADDITIONAL OPTIONS



ROHO is proud to offer a wide variety of seating solutions.

Discover additional cushion options that do not incorporate ROHO's DRY FLOATATION® Technology. ROHO offers lightweight and cost effective solutions for almost any individual.

ROHO® AirLITE® Cushion

The AirLITE Cushion is designed to improve transfers by having a lower front and middle section, offering positioning capabilities with leg troughs. The lower height of the cushion also contributes to a greater feeling of security in the wheelchair.

A non-adjustable ROHO AIR FLOATATION™ air insert is built into the cushion under the ischials, to help prevent the individual from bottoming out. Softer, more comfortable foam allows the cushion to conform better to the individual's physiology.

Based on clinical, scientific and/or engineering evidence, this product may be suitable for individuals who:

- Are at risk for skin/soft tissue breakdown
- Have normal sensation
- Require increased stability
- Require lower extremity alignment
- Need a lightweight, non-adjustable seating system



*Cover is required for use.

DIMENSIONS

Width: 13" to 20" / 33 cm to 51 cm
 Height: Approx. 2" to 3.75" / 5 cm to 9.5 cm
 Weight: 2.25 lbs. / 1.0 kg

WEIGHT LIMIT

300 lbs. / 136 kg

LIMITED WARRANTY

Cushion: 24 Months
 Cover: 6 Months

SKIN PROTECTION POSITIONING ADJUSTABILITY ADAPTABILITY COMFORT



ROHO® MOSAIC® Cushion

The MOSAIC Cushion is made of 2.75" / 7 cm DEHP-free polyvinylchloride interconnected air cells that are easy to inflate and adjust. This lightweight cushion is designed to provide a comfortable, functional and stable sitting environment. The MOSAIC cushion is perfectly suited for individuals who require skin and soft tissue protection, support and comfort. Using AIR FLOATATION™ Technology, the interconnected air cells distribute forces evenly across the cushion and allow it to be used effectively for skin and soft tissue protection.

Based on clinical, scientific or engineering evidence, this product may be suitable for individuals who:

- Are at risk for skin/soft tissue breakdown
- Currently have skin/soft tissue breakdown (lesser stage pressure ulcer following an individual assessment)
- Have normal or impaired sensation
- Require accommodation of pelvic asymmetry, up to approximately 1 in. / 2.5 cm

DIMENSIONS

Width: 16" to 18" / 40.5 cm to 45.5 cm
 Height: 2.75" / 7 cm
 Weight: Approx. 0.75 lbs. / 0.4 kg

WEIGHT LIMIT

250 lbs. / 113 kg

LIMITED WARRANTY

Cushion: 12 Months
 Cover: 6 Months

SKIN PROTECTION POSITIONING ADJUSTABILITY ADAPTABILITY COMFORT



ROHO® SOFFLEX® Cushion

The SOFFLEX Cushion, constructed of urethane, is an adjustable, air-filled wheelchair support surface and is intended to conform to an individual's seated shape to provide skin/soft tissue protection.

The SOFFLEX Cushion is recommended for seated individuals who may:

- Have a history of or currently have skin/soft tissue breakdown
- Are at risk for skin/soft tissue breakdown
- Have normal or impaired sensation in the seated area
- Require comfort and support
- Have fair to good seated balance
- Require accommodation of pelvic asymmetry, up to approximately 1 in. / 2.5 cm

DIMENSIONS

Width: 16.25" to 20.25" / 41.5 cm to 51.5 cm
 Height: 3" / 7.5 cm
 Weight: .75 lbs. / 0.3 kg

WEIGHT LIMIT

300 lbs. / 136 kg

LIMITED WARRANTY

Cushion: 18 Months
 Cover: 6 Months

SKIN PROTECTION POSITIONING ADJUSTABILITY ADAPTABILITY COMFORT

LTV SEAT® CUSHION

The LTV cushion is a non-medical cushion application for anyone who sits for extended periods of time. The LTV Seat Cushion is lightweight and easy to fold, making this the perfect cushion to take anywhere. Shape Fitting Technology® provides unsurpassed comfort. The LTV Seat Cushion is made of individual, interconnected air cells that allow air to slowly transfer from chamber to chamber and evenly distribute body weight pressure.

The LTV cushion is available in one size. Two cover options are available: Quilted fabric or Ultra Leather™.

The LTV Cushion is great for:

- Long car rides
- Airplane travel
- Stadium/event seating
- Home seating
- Office seating
- Hospital guest chairs



DIMENSIONS

Width: 18" / 45.5 cm

Depth: 16" / 40.5 cm

Height: Adjustable 1" to 2" / 2.5 cm to 5.5 cm

Weight: 1.0 lbs. / 0.2 kg

WEIGHT LIMIT

250 lbs. / 133 kg

LIMITED WARRANTY

Cushion: 12 Months

Cover: 6 Months

SKIN PROTECTION POSITIONING ADJUSTABILITY ADAPTABILITY COMFORT

SEAT CUSHIONS CUSHION ACCESSORIES



If you are looking for replacement parts or accessories for your ROHO – discover all the options that are available on ROHO.com.

Need a place for your wallet, keys, glasses or phone?

Now you have the option to buy a cushion cover with a pocket! Pockets are available for any style cover. Contact your ROHO Distributor or contact Customer Care at (618) 277-9150 to order your cushion cover with a pocket today!



ROHO® Standard and SELECT Series Cushion Replacement Covers

The ROHO Standard and SELECT Series Cushion Covers are intended to protect the cushion. Features include a two-way stretch top, spacer fabric sides with a zipper; and a non-skid bottom with hook and loop fasteners to help secure the cushion in the chair.

This product is suitable for individuals who are using a compatible ROHO cushion.



ROHO® Heavy Duty Cushion Covers

The ROHO Heavy Duty Cushion Cover is a fluid-resistant cover intended to protect the cushion. Features include a two-way stretch, fluid-resistant top and sides with a zipper; and a non-skid bottom with hook and loop fasteners to help secure the cushion in the chair.

The ROHO Heavy Duty Cushion Cover is recommended for individuals who:

- Are using a compatible ROHO cushion
- Seek additional incontinence protection for cushion
- Seek ozone protection for cushion
- Seek additional resistance to everyday wear and tear



ROHO Standard Cushion Covers and Heavy Duty Cushion Covers are available for the following cushions:

ROHO® Single or Dual Compartment:

HIGH PROFILE®
 MID PROFILE™
 LOW PROFILE®
 HIGH PROFILE® Sensor Ready™ Cushion
 MID PROFILE™ Sensor Ready™ Cushion
 ENHANCER®
 MOSAIC®

ROHO® Select Cushion Cover for:

Note: The Select Cushion Covers include an opening for the ISOFLO® Control.

QUADTRO SELECT® HIGH PROFILE®
 QUADTRO SELECT® MID PROFILE™
 QUADTRO SELECT® LOW PROFILE®
 CONTOUR SELECT®

Replacement covers are available for the following cushions:

Hybrid Elite® Cushion Cover
 HARMONY® Cushion Cover
 nexus SPIRIT® Cushion Cover
 MINI-MAX® Cushion Cover
 PACK-IT® Cushion Cover
 SOFFLEX® Cushion
 AirLite® Cushion
 LTV Seat® Cushion



ROHO® Planer Solid Seat Base

The ROHO Planer Solid Seat Base provides a solid insert for use with ROHO cushioning products. The base provides support to assist with positioning. Made of polyurethane coated birch plywood, the 3/8 inch / 1 centimeter base easily fits under the cushion, inside the ROHO cushion cover.

- Eliminates the hammock effect of sling seats.

LIMITED WARRANTY
12 months



ROHO® Contour Base

The lightweight, molded base provides a solid foundation for sling seat wheelchairs. Easily fits under the cushion inside the ROHO® cushion cover. The base provides support to assist with positioning.

LIMITED WARRANTY
12 months

Visit ROHO.com for product information, including available sizes.



ROHO® Privacy Shield

The ROHO Privacy Shield is perfect for anyone who wants coverage between the bottom of the back of their wheelchair to the seat base of their wheelchair.

Unlike other privacy shields on the market, the ROHO Privacy Shield is designed to attach directly to the canes of a wheelchair, so it will work with most wheelchairs. It is easy to attach and easy to detach for cleaning.

LIMITED WARRANTY

6 months



Hand Pump, Single or Dual Action

ROHO Hand Inflation Pumps are compatible with all air-filled ROHO medical and LTV® products.





SPECIALTY APPLICATION PRODUCTS

SPECIALTY APPLICATION PRODUCTS



Pressure ulcers can happen anywhere the body touches a surface where tissues are deformed and blood flow is limited.

ROHO has developed products to keep our customers comfortable and protect individuals from developing pressure ulcers in almost any situation. Discover all the options available at [ROHO.com](https://www.roho.com).

ROHO® Custom Cushions

ROHO will customize a cushion to fit almost any surface. It is important that anyone who needs protection is covered. ROHO cushions can be specifically designed to meet an individual's size, diagnosis or specific need.

DRY FLOATATION® cushions can be custom designed for an individual with any combination of the following techniques:

- Modify number of valves / compartments
- Modify cell heights
- Add or delete rows of cells beyond dimension of standard size models
- Remove cells or rows
- Add ventilation/drainage holes through base of cushion

Custom Cushions are designed and priced upon request. Restrictions may apply. If you have any questions or are looking for something in particular, please contact your ROHO Distributor or contact Customer Care at cc@roho.com, or call (618) 277-9150.

The Design-A-ROHO worksheet is also available on ROHO.com.



DIMENSIONS

Width: Custom
Height: Custom
Weight: Custom

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

Cushion: 24 Months
Cover: 6 Months



ROHO® Shower / Commode Cushion

The ROHO Shower/Commode Cushion is an adjustable, air-filled, cellular-design, support surface that utilizes DRY FLOATATION® Technology and is intended for short-term cushioning needs on bedside commodes and shower chairs to provide skin and soft tissue protection. The Shower/Commode Cushion is easy to clean and disinfect. Cover not included.

Visit ROHO.com for size information.

DIMENSIONS

Height: 2.25" / 5.5 cm
 Width: 14.75" / 37.5 cm
 Depth: 14.75" / 37.5 cm
 Weight: 1.5 lbs. / 0.7 kg

WEIGHT LIMIT

None, if properly sized to the individual

LIMITED WARRANTY

24 months



ROHO® Toilet Seat Cushion

The ROHO Toilet Seat Cushion is an adjustable, air-filled, cellular-design, neoprene surface intended for short-term cushioning needs on toilets and portable bedside commodes. The Toilet Seat Cushion snaps easily into place. It is easy to clean and disinfect. Cover not included.

DIMENSIONS

Height: 2.25" / 5.5 cm
 Width: 7.75" / 20 cm
 Depth: 15.75" / 40 cm
 Weight: 1.25 lbs. / 0.5 kg

WEIGHT LIMIT

None, if properly sized to the individual

LIMITED WARRANTY

24 months



ROHO® HEAL PAD® Cushion


The HEAL PAD offers an adjustable therapeutic environment, which may assist in the prevention and healing of the heel. The heel pad helps reduce shear and friction and provides support for these high-risk pressure points. HEAL PADS are available in single or pair options.

DIMENSIONS

Height: 1.5" / 4 cm
 Width: 11.5" / 29.5 cm
 Depth: 12.5" / 31.5 cm
 Weight: 0.75 lbs. / 0.96 kg

LIMITED WARRANTY

24 months

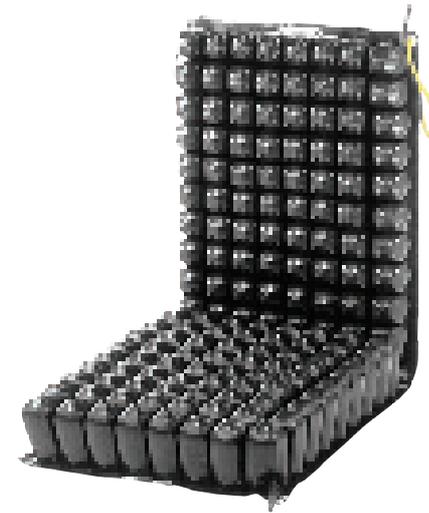

ROHO® Recliner System

The ROHO Recliner System is an adjustable, air-filled, cellular-design, support surface that utilizes DRY FLOATATION® Technology. It is intended to conform to an individual's seated shape to provide localized skin/soft tissue protection and an environment to facilitate wound healing.

The Recliner System can be specifically measured and designed to the dimensions of an individual's seating surface. The Recliner System provides back and seating surface protection.

The Recliner System can be used for almost any support surface such as a home recliner, a reclining wheelchair or a hospital bed-side chair.

DIMENSIONS

Height: varies by size
 Width: varies by size
 Depth: varies by size
 Weight: varies by size

WEIGHT LIMIT

None, if properly sized to the individual.

LIMITED WARRANTY

24 months



THE ADAPTOR® PAD

The ADAPTOR Pad can be used as an interface on virtually any flat or contoured surface to protect soft tissues. The ADAPTOR Pad is not intended as a seating support surface; it is especially useful when it is cut to fit various surface shapes, such as arm rests, trays, head and foot rests or anywhere uncontrolled movement may need protection.

- Individually sealed air cells
- Can be cut to size
- Absorbs shock
- Protects against shear and friction
- Enhances soft tissue protection

DIMENSIONS

Height: 1" / 2 cm
 Width: 9.75" / 25 cm
 Depth: 13.5" / 34 cm
 Weight: 0.5 lbs. / 0.2 kg

WEIGHT LIMIT

None, if properly sized to the individual.

LIMITED WARRANTY

24 months



ROHO® MINI-MAX® Cushion

The ROHO MINI-MAX Cushion is an adjustable, air-filled, cellular-design surface intended as a supplemental seating surface. The cushion provides significant shock impact absorption and skin protection for short term needs, such as for athletic competition, bathing, traveling, or as a scapular, sacral or lumbar support.

DIMENSIONS

Height: 1.25" / 3 cm
 Width: 15" to 18.25" / 38.5 cm to 46.5 cm
 Weight: 2 lbs. / .9 kg

WEIGHT LIMIT

None, if properly sized to the individual.

LIMITED WARRANTY

Cushion: 24 months
 Cover: 6 months

ROHO® PACK-IT® Cushion

The ROHO PACK-IT is designed for short-term comfort and soft tissue protection needs. The PACK-IT is a smaller version of the MINI-MAX Cushion and is excellent for athletic competition, bathing, traveling, or as a scapular, sacral or lumbar support. The PACK-IT cover includes a strap to allow for self-attachment.



DIMENSIONS

Height: 1.25" / 3.0 cm
Width: 16.25" / 41.5 cm
Depth: 9.5" / 24.0 cm
Weight: 1.5 lbs. / 0.7 kg

WEIGHT LIMIT

None, if properly sized to the individual.

LIMITED WARRANTY

Cushion: 24 months
Cover: 6 months

ROHO® Lumbar Support Cushion

The ROHO Lumbar Support Cushion is an adjustable, air-filled, cellular-design surface that utilizes DRY FLOATATION® Technology and is intended to conform to an individual's lumbar region to provide support. The Lumbar Support Cushion is adjustable to the individual for maximum positioning and easily attaches with hook and loop fasteners. Lumbar Support Cushion is recommended for individuals who are at risk for skin/soft tissue breakdown, want or need lumbar support and/or protection, and/or require options for use beyond a wheelchair such as an office chair, recliner, etc.



DIMENSIONS

Height: 4.25" / 10.5 cm
Width: 15.25" / 39 cm
Depth: 6.5" / 16 cm
Weight: 1 lbs. / 0.5 kg

LIMITED WARRANTY

24 months

ROHO® Thyroid Pillow

The Thyroid Pillow is designed for individuals undergoing surgical procedures which may require hyperextension of the neck. The pillow assists by holding the neck securely in place during procedures and the pillow may be adjusted to match the individual's shape. One size pillow accommodates most individuals.



DIMENSIONS

Height: 5.5" / 14 cm
Width: 15" / 38 cm
Depth: 8.5" / 21.5 cm

LIMITED WARRANTY

24 months

Can't find what you're looking for?

Visit ROHO.com to:

- Locate a distributor
- Find more product information
- Learn more about the science and technology behind ROHO products
- Register a ROHO product
- Read frequently asked questions
- Watch how-to videos

ROHO Customer Care Department can also be reached by emailing cc@roho.com or by calling (618) 277-9150.



WHEELCHAIR BACKS

WHEELCHAIR BACKS

ROHO® AGILITY® BACK SYSTEM



ROHO's AGILITY product line leads the market in comfort and strength for positioning backrests.

After successfully supporting the wheelchair seating and positioning market for over 40 years, ROHO knew there were more opportunities to meet the needs of clinicians and end users with an innovative positioning back system.

The ROHO AGILITY back is designed to accommodate various body sizes and shapes. It can be easily adjusted – even when the individual is seated. Adaptable to virtually any wheelchair, the AGILITY back will stay secure and in position. AGILITY backs are extremely light weight and durable. The back shell is composed of aircraft-grade aluminum with durable, adjustable AIR FLOATATION™ inserts to provide unparalleled comfort.

Replacement covers are available for all ROHO back systems. Contact ROHO Distributor or ROHO Customer Care for additional information.

AGILITY® Hardware


The AGILITY hardware is made from Zytel®, a product of DuPont™, which provides an excellent strength-to-weight ratio. Zytel is valued for its high mechanical strength and abrasion resistance, as well as chemical and flame resistant properties.

The hardware exhibited no damage and did not move out of position when subjected to 300 lbs. / 136 kg of force exerted for 100,000 cycles directly onto the most vulnerable part of the back. AGILITY sustained over 1,000x more cycles than its leading competitor.

AGILITY backs are available with two adjustable hardware attachment options. Both options can be adjusted the same way once installed.


Quick Release Hardware

Quick release hardware makes removal for transport or storage a snap, even for those with limited hand dexterity. Simply unlock and lift by the handle.

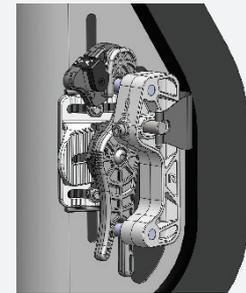

Fixed Hardware

Fixed hardware is ideal for those who rarely need to remove their back and want a smaller mount.

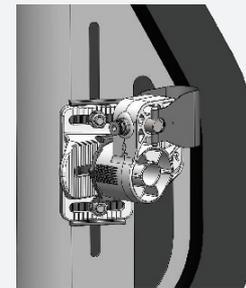

ROHO® AIR FLOATATION™ Technology

ROHO AGILITY backs are designed to be lightweight and to conform to the user's body. AGILITY backs integrate ROHO's AIR FLOATATION™ Technology, which is fully adjustable to tailor air volume to comfort level and support postural changes.



Flip-a-bility

- Fully adjustable to fit more body shapes.
- Can be “flipped” to accommodate wider torsos or broader shoulders.
- Full range of angle, depth, width and vertical adjustment capabilities are available to address specific positioning needs.





AGILITY® Minimum Contour

The AGILITY Minimum Contour back system provides basic postural support for independent wheelchair users who have good trunk control and need adjustability to enhance range of motion and mobility.

Depth of overall back system:
 Foam only: 2.5" / 6.5 cm
 With Air Insert: 3" / 7.5 cm

Available Models:

- Foam + Lumbar Support
- Foam with Air Insert + Lumbar Support

DIMENSIONS

Width: 13" to 20" / 33 cm to 51 cm
 Height: 10" to 18" / 25.5 cm to 45.5 cm
 Weight: 2.5 lbs. to 5.25 lbs. / 1.1 kg to 2.3 kg

WEIGHT LIMIT

300 lbs / 136 kg

LIMITED WARRANTY

Back Shell &
 Hardware: 36 months
 Air Insert: 18 months
 Cover & Foam: 18 months



AGILITY® Mid Contour

The AGILITY Mid Contour back system is recommended for individuals who require postural support, are somewhat dependent with transfers and need minimal trunk support.

Depth of overall back system:
 With Air Insert: 4" / 10 cm

Available Models:

- Foam with Air Insert + Lumbar Support

DIMENSIONS

Width: 13" to 20" / 33 cm to 51 cm
 Height: 10" to 20" / 25.5 cm to 51 cm
 Weight: 2.5 lbs. to 6 lbs. / 1.1 kg to 2.8 kg

WEIGHT LIMIT

300 lbs / 136 kg

LIMITED WARRANTY

Back Shell &
 Hardware: 36 months
 Air Insert: 18 months
 Cover & Foam: 18 months



AGILITY® Max Contour

The AGILITY Max Contour back system provides a significant amount of postural support. The Max Contour is recommended for individuals who have, or have potential for, shoulder or back pain and want increased comfort as well as trunk support and control.

Depth of overall back system:
With Air Insert: 7" / 18 cm

Available Models:

- Foam with Air Insert + Lumbar Support

DIMENSIONS

Width: 13" to 20" / 33 cm to 51 cm
Height: 16" to 20" / 40.5 cm to 51 cm
Weight: 4.25 lbs. to 6.5 lbs. / 2.0 kg to 3.0 kg

WEIGHT LIMIT

300 lbs / 136 kg

LIMITED WARRANTY

Back Shell &
Hardware: 36 months
Air Insert: 18 months
Cover & Foam: 18 months

*All Max models include adjustable lateral air pads



AGILITY® Custom

Design an adjustable DRY FLOATATION® insert that's individually tailored to the user's unique body shape and postural support needs with ROHO's exclusive AGILITY Custom back system.

AGILITY Custom backs can be fitted to all AGILITY back shell sizes. Custom backs are available in consistent or variable DRY FLOATATION Technology air cells for superior comfort and support. Multiple or single air valve placements can also be specified for air flow control.

Visit ROHO.com, call Customer Care at (618) 277-9150 or contact your local distributor for more information.

LIMITED WARRANTY

Back Shell &
Hardware: 36 months
Air Insert: 36 months
Cover & Foam: 18 months

Fixed Lateral



Swing-away Lateral

WEIGHT LIMIT

300 lbs. / 136 kg

LIMITED WARRANTY

Brackets: 36 months
Sleeve, cover, foam and base:
6 months

AGILITY® Laterals

AGILITY Laterals are designed to provide additional trunk support and positioning to the individual. Laterals are available in swing-away and fixed brackets with flush or offset mounts. Individuals may order in single quantities to allow combination of one fixed and one swing-away, which allows for easier transfers.

Swing-away laterals:

Allow individuals to transfer more easily.

Available in flush, 1/2" / 1.5 cm, 1" / 2.5 cm and 2" / 5.0 cm offset.

Brackets come with a curved base, foam pad and cover.

Fixed laterals:

Available in flush and 1/2" / 1.5 cm offset.

Brackets come with a flat base, foam pad and cover.

The AGILITY Laterals are designed for use only with ROHO AGILITY Minimum Contour Back Systems and ROHO AGILITY Mid Contour Back Systems.

AGILITY® Replacement Covers

Exclusive AGILITY replacement covers are made out of durable, vapor-permeable polyester that is specially designed for breathability. The cover is designed to help prevent foam from moving out of position. Covers are washable and available for:

- AGILITY Minimum Contour Back
- AGILITY Mid Contour Back
- AGILITY Max Contour Back



LIMITED WARRANTY
18 months

AGILITY® Headrest

Compatible with any AGILITY Back System, the Headrest by Therafin® is a versatile headrest that is completely adjustable in any direction. It adjusts horizontally, vertically, and pivots on a ball. Positioning stop collar allows you to remove the headrest assembly without changing the desired headrest position.



AGILITY® Headrest Adaptor Plate

The ROHO AGILITY Headrest Adapter Plate is hardware intended to be used with ROHO AGILITY Back Systems to attach commercially-available wheelchair headrests. The AGILITY Headrest Adapter Plate is designed to be compatible with OEM headrests. Go to ROHO.com for more detailed specifications.







MATTRESS OVERLAYS

MATTRESS OVERLAYS



ROHO's mattress overlays are the perfect solution for skin and soft tissue protection in hospitals, long-term care or home care settings.

The interconnected, air filled cells conform to the contours of the body, to maintain the shape which minimizes tissue deformation to enhance the healing process, and prevent tissue breakdown.

Non-powered support surfaces provide individuals and caregivers with peace-of-mind during power outages or in any situation where power is not easily accessible.

ROHO® DRY FLOATATION® Mattress Overlay

The ROHO DRY FLOATATION Mattress Overlay System creates the ideal environment of protection and healing with the ORIGINAL non-powered, air-filled, adjustable, zoned, reactive mattress overlay.

The performance, efficiency and quality of this overlay system are supported by dozens of clinical case studies. The four sections of the mattress overlay can be independently adjusted, providing a low-pressure environment over the entire contact area of the body. The DRY FLOATATION Mattress Overlay is offered with an enclosure cover or a reusable cover.

ROHO DRY FLOATATION Bariatric Kit add-on sections are available to work in conjunction with a standard size ROHO DRY FLOATATION Mattress Overlay to widen the mattress. Covers must be ordered separately for Bariatric kit sections.



DIMENSIONS

Width: 33.75" / 86 cm

Height: 3.25" / 8.5 cm

Depth: 75.0" / 190.5 cm

Weight: 32.0 lbs. / 14.5 kg

WEIGHT LIMIT

No weight limit, yet the cushion must be properly sized to the individual.

LIMITED WARRANTY

24 Months



ROHO® SOFFLEX® 2 Mattress Overlay System

The ROHO SOFFLEX 2 Mattress Overlay is a cost effective solution that provides superior skin/soft tissue protection.

The SOFFLEX 2 Mattress Overlay is a non-powered, air-filled, adjustable, zoned, reactive, full support surface that is intended to conform to an individual's shape to provide skin/soft tissue and deep tissue injury protection and an environment to facilitate wound healing. The SOFFLEX 2 Mattress Overlay provides individualized pressure, shear and friction management through force redistribution.

The SOFFLEX 2 Mattress Overlay is offered with an enclosure cover or a reusable cover.

DIMENSIONS


Width: 36" / 91.5 cm
Height: 3.5" / 9 cm
Depth: 81.75" / 207.5 cm
Weight: 6.5 lbs. / 2.9 kg

WEIGHT LIMIT

330 lbs. / 150 kg

LIMITED WARRANTY

Mattress Overlay:
18 months
Cover: 6 months

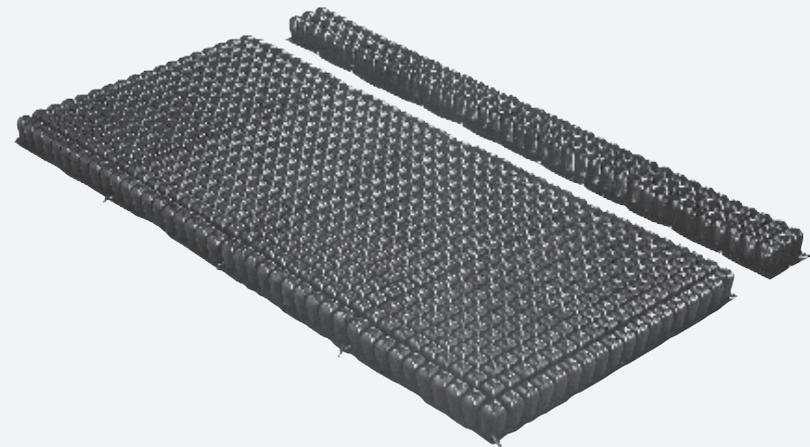

Custom DRY FLOATATION® Mattress Overlay

ROHO offers Custom DRY FLOATATION Overlays for almost any application. Many individuals need skin and deep tissue protection on any mattress surface that they will be lying on for extended periods of time.

If the standard ROHO DRY FLOATATION Mattress Overlay sizes do not align with the individual's needs, contact your ROHO Distributor or ROHO Customer Care at (618) 277-9150 for more information.

Create a custom overlay for:

- Bariatric sizes
- Pediatric sizes



PRODIGY® Mattress Overlay System

The ROHO® PRODIGY® Mattress Overlay System provides an individualized fit in order to protect skin and soft tissue from breakdown.

A non-powered, air-filled, adjustable, zoned, reactive, full support surface, PRODIGY is intended to conform to an individual's shape to provide skin/soft tissue and deep tissue injury protection. Flexible, interconnected air cells can be adjusted to match the individual's body contour. Each of the three sections can be adjusted separately to address specific, changing needs.

- Zoned - Three sections each 36 in. x 27.5 in. / 91.5 cm x 70 cm.
- Segmented for use in positions other than 180°.
- Hydrostatic distribution of load across the entire contact area.
- Provides 3 in. / 7.5 cm of immersion.

DIMENSIONS

Width: 36" / 91.5 cm
Height: 3.5" / 9 cm
Depth: 81.75" / 207.5 cm
Weight: 9.5 lbs. / 4.3 kg

WEIGHT LIMIT

300 lbs. / 136 kg

LIMITED WARRANTY

12 Month Limited
Warranty on soft goods.

ROHO offers the total seating solution for skin and soft tissue protection, whether it's in a home, hospital or long-term care setting.

If it's not a ROHO, it's a risk.



Special Applications

ROHO DRY FLOATATION O.R. Pad
ROHO DRY FLOATATION Head/Neck Pillow
ROHO DRY FLOATATION Nesting Pad
ROHO DRY FLOATATION Isolette





RESOURCES

ROHO® Cushion Sizing Chart

Standard DRY FLOATATION® Cushion Sizing Chart for:

- ROHO® LOW PROFILE® Single or Dual Compartment Cushion
- ROHO® MID PROFILE™ Single or Dual Compartment Cushion
- ROHO® HIGH PROFILE® Single or Dual Compartment Cushion
- ROHO® ENHANCER® Cushion
- ROHO® MID PROFILE™ Single Compartment with Sensor Ready™ Technology
- ROHO® HIGH PROFILE® Single Compartment with Sensor Ready™ Technology

ROHO SELECT Series Cushion Sizing Chart for:

- ROHO® QUADTRO SELECT® LOW PROFILE® Cushion
- ROHO® QUADTRO SELECT® MID PROFILE™ Cushion
- ROHO® QUADTRO SELECT® HIGH PROFILE® Cushion
- ROHO® CONTOUR SELECT® Cushion

Select the correct size ROHO Cushion for your wheelchair

Step 1

Measure the wheelchair seat width and depth.

Note: Wheelchair manufacturers' published seat dimensions may vary.

Step 2

Use your wheelchair seat dimensions and the below table to help you determine the number of cells in cushion width and in depth. The current version is available at ROHO.com.

ROHO® Neoprene Cushion Sizing Chart

Number of Cells	Recommended Chair Size (cm)	Recommended Chair Size (Inches)
5	22 to 26.5	10
6	26.5 to 31	11 - 12
7	31 to 35	13
8	35 to 39.5	14 - 15
9	39.5 to 44	16 - 17
10	44 to 48.5	18 - 19
11	48.5 to 52.5	20 - 21
12	52.5 to 57	22
13	57 to 61	23 - 24
14	61 to 65.5	25
15	65.5 to 70	26 - 27

Note: Availability of cushion sizes varies by model.

DRY FLOATATION® Cushion Model Numbers:

- 1** Select the appropriate prefix for the chosen product.

QUADTRO SELECT Cushion - QS	ENHANCER Cushion - ENH
Single Compartment Cushion - 1R	CONTOUR SELECT Cushion - CS
Dual Compartment Cushion - 2R	
- 2** Select the appropriate number of cells and place behind the prefix selected in previous step.

For example: QS88 (QUADTRO SELECT cushion: 8 cells wide and 8 cells deep), 1R99 (Single Compartment Cushion: 9 cells wide and 9 cells deep).
- 3** If applicable, determine what cell height: MID PROFILE (MP) or LOW PROFILE (LP). Do not list the cell height for HIGH PROFILE Cushions.

For example: QS88MP (QUADTRO SELECT MID PROFILE), 1R99LP (Single Compartment LOW PROFILE)
- 4** Determine the cover type between a Standard or Select Series Cover (C) or a Heavy Duty Cover (HD).

For example: QS88MPC (QUADTRO SELECT MID PROFILE with a Select Series Cover), 1R99LPHD (Single Compartment LOW PROFILE with a Heavy Duty Cover)

Single Compartment Cushions with Sensor Ready™ Technology Model Numbers:

- 1** The prefix for Single Compartment Cushions with Sensor Ready Technology is 1R.
- 2** Select the appropriate number of cells (see chart) and place behind the prefix selected in previous step.


For Example: 1R99 (Single Compartment Cushion with Sensor Ready™ Technology: 9 cells wide and 9 cells deep).
- 3** Determine what cell height: HIGH PROFILE (H) or MID PROFILE (M).

For example: 1R99M (Single Compartment MID PROFILE Cushion)
- 4** Add a dash and determine the cover type between Standard Cover (C) or a Heavy Duty Cover (CH).


For example: 1R99M-CA (Single Compartment MID PROFILE Cushion with a Standard Cover)
- 5** Confirm that this is a Sensor Ready Cushion by adding another dash with Sensor Ready (SR).
 High Profile - Width: 11.5" to 26.75" / 29 cm to 68 cm
 Mid Profile - Width: 13.25" to 20" / 33.5 cm to 50.5 cm
 Special sizes not available.

For example: 1R99M-CA-SR (MID PROFILE Cushion with Sensor Ready™ Technology with a Standard Cover)

Standard DRY FLOATATION Cushions

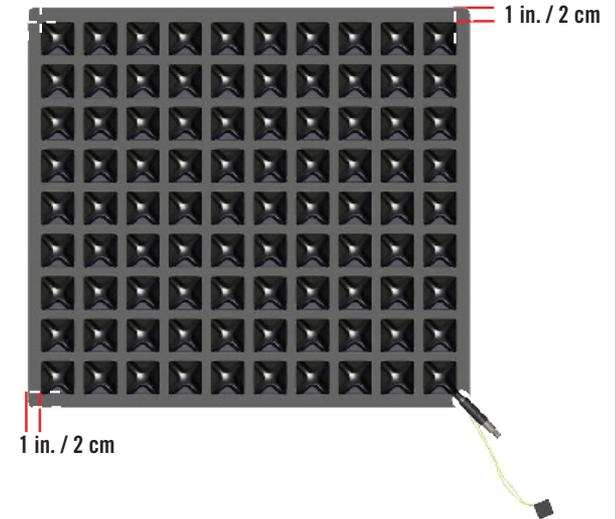

SELECT Series Cushions

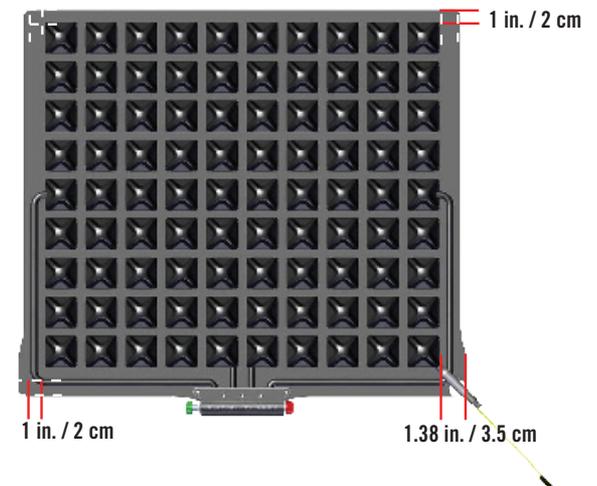

Cushion Sizing Charts

Hybrid Elite® Single Valve Cushion

Model	Fits Chair Size
1RHY1414C	14" x 14" / 36 cm x 36 cm
1RHY1415C	14" x 15" / 36 cm x 38 cm
1RHY1416C	14" x 16" / 36 cm x 41 cm
1RHY1515C	15" x 15" / 38 cm x 38 cm
1RHY1516C	15" x 16" / 38 cm x 41 cm
1RHY1517C	15" x 17" / 38 cm x 43 cm
1RHY1615C	16" x 15" / 41 cm x 38 cm
1RHY1616C	16" x 16" / 41 cm x 41 cm
1RHY1617C	16" x 17" / 41 cm x 43 cm
1RHY1618C	16" x 18" / 41 cm x 46 cm
1RHY1620C	16" x 20" / 41 cm x 51 cm
1RHY1716C	17" x 16" / 43 cm x 41 cm
1RHY1717C	17" x 17" / 43 cm x 43 cm
1RHY1718C	17" x 18" / 43 cm x 46 cm
1RHY1816C	18" x 16" / 46 cm x 41 cm
1RHY1817C	18" x 17" / 46 cm x 43 cm
1RHY1818C	18" x 18" / 46 cm x 46 cm
1RHY1820C	18" x 20" / 46 cm x 51 cm
1RHY2017C	20" x 17" / 51 cm x 43 cm
1RHY2018C	20" x 18" / 51 cm x 46 cm
1RHY2020C	20" x 20" / 51 cm x 51 cm
1RHY2218C	22" x 18" / 56 cm x 46 cm
1RHY2220C	22" x 20" / 56 cm x 51 cm
1RHY2418C	24" x 18" / 61 cm x 46 cm
1RHY2420C	24" x 20" / 61 cm x 51 cm

nexus SPIRIT® Cushion

Model	Fits Chair Size
NS1414C	14" x 14" / 36cm x 36cm
NS1416C	14" x 16" / 36cm x 41cm
NS1515C	15" x 15" / 38cm x 38cm
NS1616C	16" x 16" / 41cm x 41cm
NS1618C	16" x 18" / 41cm x 46cm
NS1717C	17" x 17" / 43cm x 43cm
NS1816C	18" x 16" / 46cm x 41cm
NS1818C	18" x 18" / 46cm x 46cm
NS2018C	20" x 18" / 51cm x 46cm

Hybrid Elite® Dual Valve Cushion

Model	Fits Chair Size
2RHY1414C	14" x 14" / 36 cm x 36 cm
2RHY1415C	14" x 15" / 36 cm x 38 cm
2RHY1416C	14" x 16" / 36 cm x 41 cm
2RHY1515C	15" x 15" / 38 cm x 38 cm
2RHY1516C	15" x 16" / 38 cm x 41 cm
2RHY1517C	15" x 17" / 38 cm x 43 cm
2RHY1615C	16" x 15" / 41 cm x 38 cm
2RHY1616C	16" x 16" / 41 cm x 41 cm
2RHY1617C	16" x 17" / 41 cm x 43 cm
2RHY1618C	16" x 18" / 41 cm x 46 cm
2RHY1620C	16" x 20" / 41 cm x 51 cm
2RHY1716C	17" x 16" / 43 cm x 41 cm
2RHY1717C	17" x 17" / 43 cm x 43 cm
2RHY1718C	17" x 18" / 43 cm x 46 cm
2RHY1816C	18" x 16" / 46 cm x 41 cm
2RHY1817C	18" x 17" / 46 cm x 43 cm
2RHY1818C	18" x 18" / 46 cm x 46 cm
2RHY1820C	18" x 20" / 46 cm x 51 cm
2RHY2017C	20" x 17" / 51 cm x 43 cm
2RHY2018C	20" x 18" / 51 cm x 46 cm
2RHY2020C	20" x 20" / 51 cm x 51 cm
2RHY2218C	22" x 18" / 56 cm x 46 cm
2RHY2220C	22" x 20" / 56 cm x 51 cm
2RHY2418C	24" x 18" / 61 cm x 46 cm
2RHY2420C	24" x 20" / 61 cm x 51 cm

HARMONY® Cushion

Model	Fits Chair Size
H1416C	14" x 16" / 36cm x 41cm
H1616C	16" x 16" / 41cm x 41cm
H1618C	16" x 18" / 41cm x 46cm
H1716C	17" x 16" / 43cm x 41cm
H1718C	17" x 18" / 43cm x 46cm
H1816C	18" x 16" / 46cm x 41cm
H1818C	18" x 18" / 46cm x 46cm
H2018C	20" x 18" / 51cm x 46cm

Note: Sizes listed represent chair size. Actual cushion size is approximately 1.5" deeper to allow for placement between back support canes.

Cushion Sizing Charts

AirLITE® Cushion

Model	Fits Chair Size
AL1313HD	13" x 13" / 33cm x 33cm
AL1315HD	13" x 15" / 33cm x 38cm
AL1414HD	14" x 14" / 36cm x 36cm
AL1416HD	14" x 16" / 36cm x 41cm
AL1515HD	15" x 15" / 38cm x 38cm
AL1517HD	15" x 17" / 38cm x 43cm
AL1616HD	16" x 16" / 41cm x 41cm
AL1618HD	16" x 18" / 41cm x 46cm
AL1717HD	17" x 17" / 43cm x 43cm
AL1718HD	17" x 18" / 43cm x 46cm
AL1720HD	17" x 20" / 43cm x 51cm
AL1816HD	18" x 16" / 46cm x 41cm
AL1818HD	18" x 18" / 46cm x 46cm
AL1820HD	18" x 20" / 46cm x 51cm
AL1917HD	19" x 17" / 48cm x 43cm
AL1919HD	19" x 19" / 48cm x 48cm
AL2017HD	20" x 17" / 51cm x 43cm
AL2018HD	20" x 18" / 51cm x 46cm

SOFFLEX® Cushion

Model	Fits Chair Size
SFLX1616C	15.75" to 16.25" x 15.75" to 16.25" 40 cm to 41.5 cm x 40 cm to 41.5 cm
SFLX1618C	15.75" to 16.25" x 17.75" to 18.25" 40 cm to 41.5 cm x 45 cm to 46.5 cm
SFLX1717C	16.75" to 17.25" x 16.75" to 17.25" 42.5 cm to 44 cm x 42.5 to 44 cm
SFLX1816C	17.75" to 18.25" x 15.75" to 16.25" 45 cm to 46.5 cm x 40 cm to 41.5 cm
SFLX1818C	17.75" to 18.25" x 17.75" to 18.25" 45 cm to 46.5 cm x 45 cm to 46.5 cm
SFLX2018C	19.75" to 20.25" x 17.75" to 18.25" 50 cm to 51 cm x 45 cm to 46.5 cm

MOSAIC® Cushion

Model (with standard cover)	Model (with Heavy Duty cover)	Fits Chair Size
MOSAIC1616C	MOSAIC1616HD	16" x 16" / 41cm x 41cm
MOSAIC1816C	MOSAIC1816HD	18" x 16" / 46cm x 41cm 16" x 18" / 41cm x 46cm
MOSAIC1818C	MOSAIC1818HD	18" x 18" / 46cm x 46cm

Note: Sizes listed represent chair size. Actual cushion size is approximately 1.5" deeper to allow for placement between back support canes.

ROHO® AGILITY® Back System Sizing Charts

For hardware choices remove the X from the model number and replace with:
 1 for Quick Release hardware or
 2 for Fixed hardware

ROHO AGILITY Minimum Contour Back System with Air and Lumbar Pad

Model	Standard Sizes (WxH)
AG2-1410AL-X	14" x 10" / 35.5 cm x 25.5 cm
AG2-1413AL-X	14" x 13" / 35.5 cm x 33 cm
AG2-1416AL-X	14" x 16" / 35.5 cm x 40.5 cm
AG2-1418AL-X	14" x 18" / 35.5 cm x 45.5 cm
AG2-1510AL-X	15" x 10" / 38 cm x 25.5 cm
AG2-1513AL-X	15" x 13" / 38 cm x 33 cm
AG2-1516AL-X	15" x 16" / 38 cm x 40.5 cm
AG2-1518AL-X	15" x 18" / 38 cm x 45.5 cm
AG2-1610AL-X	16" x 10" / 40.5 cm x 25.5 cm
AG2-1613AL-X	16" x 13" / 40.5 cm x 33 cm
AG2-1616AL-X	16" x 16" / 40.5 cm x 40.5 cm
AG2-1618AL-X	16" x 18" / 40.5 cm x 45.5 cm
AG2-1710AL-X	17" x 10" / 43 cm x 25.5 cm
AG2-1713AL-X	17" x 13" / 43 cm x 33 cm
AG2-1716AL-X	17" x 16" / 43 cm x 40.5 cm
AG2-1718AL-X	17" x 18" / 43 cm x 45.5 cm
AG2-1810AL-X	18" x 10" / 45.5 cm x 25.5 cm
AG2-1813AL-X	18" x 13" / 45.5 cm x 33 cm
AG2-1816AL-X	18" x 16" / 45.5 cm x 40.5 cm
AG2-1818AL-X	18" x 18" / 45.5 cm x 45.5 cm
AG2-1910AL-X	19" x 10" / 48.5 cm x 25.5 cm
AG2-1913AL-X	19" x 13" / 48.5 cm x 33 cm
AG2-1916AL-X	19" x 16" / 48.5 cm x 40.5 cm
AG2-1918AL-X	19" x 18" / 48.5 cm x 45.5 cm
AG2-2010AL-X	20" x 10" / 51 cm x 25.5 cm
AG2-2013AL-X	20" x 13" / 51 cm x 33 cm
AG2-2016AL-X	20" x 16" / 51 cm x 40.5 cm
AG2-2018AL-X	20" x 18" / 51 cm x 45.5 cm

ROHO AGILITY Minimum Contour Back System with Foam and Lumbar Pad

Model	Standard Sizes (WxH)
AG2-1410FL-X	14" x 10" / 35.5 cm x 25.5 cm
AG2-1413FL-X	14" x 13" / 35.5 cm x 33 cm
AG2-1416FL-X	14" x 16" / 35.5 cm x 40.5 cm
AG2-1418FL-X	14" x 18" / 35.5 cm x 45.5 cm
AG2-1510FL-X	15" x 10" / 38 cm x 25.5 cm
AG2-1513FL-X	15" x 13" / 38 cm x 33 cm
AG2-1516FL-X	15" x 16" / 38 cm x 40.5 cm
AG2-1518FL-X	15" x 18" / 38 cm x 45.5 cm
AG2-1610FL-X	16" x 10" / 40.5 cm x 25.5 cm
AG2-1613FL-X	16" x 13" / 40.5 cm x 33 cm
AG2-1616FL-X	16" x 16" / 40.5 cm x 40.5 cm
AG2-1618FL-X	16" x 18" / 40.5 cm x 45.5 cm
AG2-1710FL-X	17" x 10" / 43 cm x 25.5 cm
AG2-1713FL-X	17" x 13" / 43 cm x 33 cm
AG2-1716FL-X	17" x 16" / 43 cm x 40.5 cm
AG2-1718FL-X	17" x 18" / 43 cm x 45.5 cm
AG2-1810FL-X	18" x 10" / 45.5 cm x 25.5 cm
AG2-1813FL-X	18" x 13" / 45.5 cm x 33 cm
AG2-1816FL-X	18" x 16" / 45.5 cm x 40.5 cm
AG2-1818FL-X	18" x 18" / 45.5 cm x 45.5 cm
AG2-1910FL-X	19" x 10" / 48.5 cm x 25.5 cm
AG2-1913FL-X	19" x 13" / 48.5 cm x 33 cm
AG2-1916FL-X	19" x 16" / 48.5 cm x 40.5 cm
AG2-1918FL-X	19" x 18" / 48.5 cm x 45.5 cm
AG2-2010FL-X	20" x 10" / 51 cm x 25.5 cm
AG2-2013FL-X	20" x 13" / 51 cm x 33 cm
AG2-2016FL-X	20" x 16" / 51 cm x 40.5 cm
AG2-2018FL-X	20" x 18" / 51 cm x 45.5 cm

Note: Fixed hardware is not available in heights taller than 16"

ROHO® AGILITY® Back System Sizing Charts

For hardware choices remove the X from the model number and replace with:
 1 for Quick Release hardware or
 2 for Fixed hardware

ROHO AGILITY Mid Contour Back System with Air and Lumbar Pad


Model	Standard Sizes (WxH)
AG3-1410AL-X	14" x 10" / 35.5 cm x 25.5 cm
AG3-1413AL-X	14" x 13" / 35.5 cm x 33 cm
AG3-1416AL-X	14" x 16" / 35.5 cm x 40.5 cm
AG3-1418AL-X	14" x 18" / 35.5 x 45.5 cm
AG3-1420AL-X	14" x 20" / 35.5 x 51 cm
AG3-1510AL-X	15" x 10" / 38 cm x 25.5 cm
AG3-1513AL-X	15" x 13" / 38 cm x 33 cm
AG3-1516AL-X	15" x 16" / 38 cm x 40.5 cm
AG3-1518AL-X	15" x 18" / 38 x 45.5 cm
AG3-1520AL-X	15" x 20" / 38 x 51 cm
AG3-1610AL-X	16" x 10" / 40.5 cm x 25.5 cm
AG3-1613AL-X	16" x 13" / 40.5 cm x 33 cm
AG3-1616AL-X	16" x 16" / 40.5 cm x 40.5 cm
AG3-1618AL-X	16" x 18" / 40.5 cm x 45.5 cm
AG3-1620AL-X	16" x 20" / 40.5 cm x 51 cm
AG3-1710AL-X	17" x 10" / 43 cm x 25.5 cm
AG3-1713AL-X	17" x 13" / 43 cm x 33 cm
AG3-1716AL-X	17" x 16" / 43 cm x 40.5 cm
AG3-1718AL-X	17" x 18" / 43 cm x 45.5 cm
AG3-1720AL-X	17" x 20" / 43 cm x 51 cm
AG3-1810AL-X	18" x 10" / 45.5 cm x 25.5 cm
AG3-1813AL-X	18" x 13" / 45.5 cm x 33 cm
AG3-1816AL-X	18" x 16" / 45.5 cm x 40.5 cm
AG3-1818AL-X	18" x 18" / 45.5 cm x 45.5 cm
AG3-1820AL-X	18" x 20" / 45.5 cm x 51 cm
AG3-1910AL-X	19" x 10" / 48.5 cm x 25.5 cm
AG3-1913AL-X	19" x 13" / 48.5 cm x 33 cm
AG3-1916AL-X	19" x 16" / 48.5 cm x 40.5 cm
AG3-1918AL-X	19" x 18" / 48.5 cm x 45.5 cm
AG3-1920AL-X	19" x 20" / 48.5 cm x 51 cm
AG3-2010AL-X	20" x 10" / 51 cm x 25.5 cm
AG3-2013AL-X	20" x 13" / 51 cm x 33 cm
AG3-2016AL-X	20" x 16" / 51 cm x 40.5 cm
AG3-2018AL-X	20" x 18" / 51 cm x 45.5 cm
AG3-2020AL-X	20" x 20" / 51 cm x 51 cm

ROHO AGILITY Max Contour Back System with Air and Lumbar Pad

Model	Standard Sizes (WxH)
AG6-1416AL-X	14" x 16" / 35.5 cm x 41 cm
AG6-1418AL-X	14" x 18" / 35.5 cm x 45.5 cm
AG6-1420AL-X	14" x 20" / 35.5 cm x 51 cm
AG6-1516AL-X	15" x 16" / 38 cm x 40.5 cm
AG6-1518AL-X	15" x 18" / 38 cm x 45.5 cm
AG6-1520AL-X	15" x 20" / 38 cm x 51 cm
AG6-1616AL-X	16" x 16" / 40.5 cm x 40.5 cm
AG6-1618AL-X	16" x 18" / 40.5 cm x 45.5 cm
AG6-1620AL-X	16" x 20" / 40.5 cm x 51 cm
AG6-1716AL-X	17" x 16" / 43 cm x 40.5 cm
AG6-1718AL-X	17" x 18" / 43 cm x 45.5 cm
AG6-1720AL-X	17" x 20" / 43 cm x 51 cm
AG6-1816AL-X	18" x 16" / 45.5 cm x 40.5 cm
AG6-1818AL-X	18" x 18" / 45.5 cm x 45.5 cm
AG6-1820AL-X	18" x 20" / 45.5 cm x 51 cm
AG6-1916AL-X	19" x 16" / 48.5 cm x 40.5 cm
AG6-1918AL-X	19" x 18" / 48.5 cm x 45.5 cm
AG6-1920AL-X	19" x 20" / 48.5 cm x 51 cm
AG6-2016AL-X	20" x 16" / 51 cm x 40.5 cm
AG6-2018AL-X	20" x 18" / 51 cm x 45.5 cm
AG6-2020AL-X	20" x 20" / 51 cm x 51 cm

Note: Fixed hardware is not available in heights taller than 16"

Cushion Comparison Chart

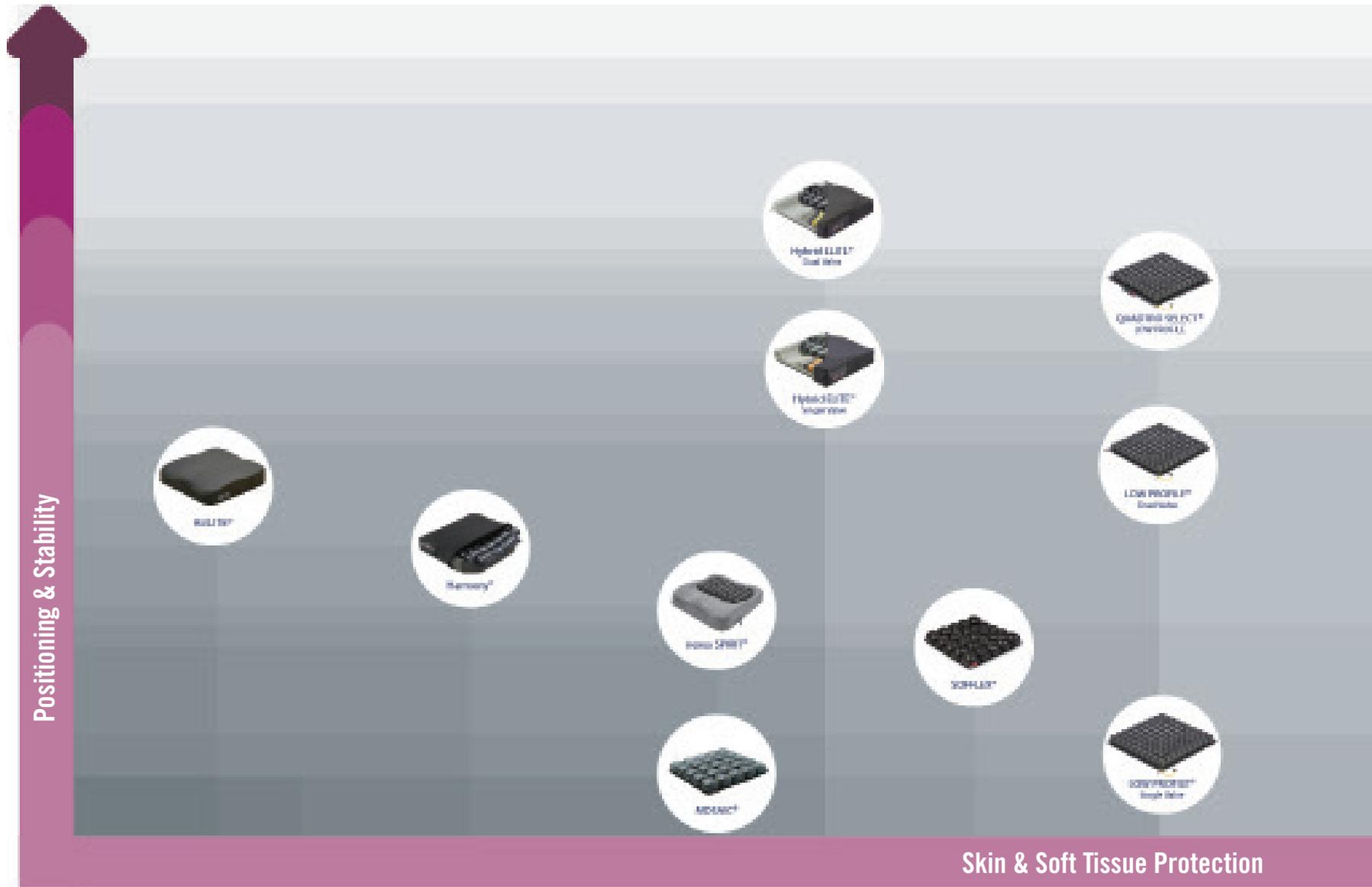



Create a complete ROHO® seating system with

ROHO® AGILITY® Back System

- STRONG
- ADJUSTABLE
- ADAPTABLE
- COMFORTABLE



ROHO® AGILITY® Minimum Contour Back System

ROHO® AGILITY® Max Contour Back System

ROHO® AGILITY® Mid Contour Back System

AGILITY® is the only aftermarket back with ROHO® AIR FLOATATION™ TECHNOLOGY inside.

Returns

All returns require prior authorization from your ROHO Distributor or ROHO, Inc. and are subject to a restocking charge. Before returning your product, contact your local Distributor or ROHO Customer Care. See our website (roho.com) for a current list of ROHO distributors.

Warranty Information

Any items sent to ROHO, Inc. without prior authorization will not be returned to the customer. Outside the U.S., contact your country's ROHO distributor. For a current list of ROHO distributors, visit our website at ROHO.com, contact ROHO International at 1-618-277-9150 or email cc@roho.com.

Be advised that a charge will be assessed for any service provided for a product that does not qualify under the product's warranty.

Note: All seat cushions, except the ADAPTOR® Pad and AirLITE® cushions, include the inflation pump, repair kit and operating instructions.

Note: All ROHO® cushions, back systems and support surfaces mentioned in this catalog come with a cover except for the following: Shower/Commode Cushion, Toilet Seat Cushion, THE ADAPTOR® Pad, Thyroid Pillow, Heal Pad® Cushion, DRY FLOATATION® O.R. Pad, DRY FLOATATION Nesting Pad, DRY FLOATATION Head/Neck Pillow, DRY FLOATATION Isolette, Lumbar Support.

Quality Management

ROHO takes quality control much further than the FDA model with its quality management system being certified to two ISO standards for over 10 years — ROHO's quality management system is currently certified to ISO 9001:2008 and ISO 13485:2003.



Intertek



Intertek



© 2016 ROHO, Inc.

ROHO, Inc. has a policy of continual product improvement and reserves the right to amend this document. The current version of this document is available at ROHO.com.

The following are trademarks and registered trademarks of ROHO, Inc.: THE ADAPTOR®, AGILITY®, AIR FLOATATION™, AIR IN PLACE®, AirLITE®, BariSelect®, CONTOUR SELECT®, DRY FLOATATION®, ENHANCER®, Fusion2K™, FusionXC™, HARMONY®, HEAL PAD®, HIGH PROFILE®, Hybrid Elite®, HybridSelect®, ISOFLO Memory Control®, LOW PROFILE®, MID PROFILE™, MINI-MAX®, MOSAIC®, nexus SPIRIT®, PACK-IT®, PRODIGY®, PROTECTYOU2™, QUADTRO SELECT®, ROHO®, Retroback®, shape fitting technology®, SOFFLEX®, SelectAir®, SELECTPROTECT®, Sensor Ready™, and Smart Check™.

Therafin® is a registered trademark of the Therafin Corporation.

Ultra Leather™ is a registered trademark of Ultra Fabrics, Inc.

Zytel® is a registered trademark of DuPont™.

With over 40 years in the market and over 90 scientific and clinical studies, **ROHO** is the efficacy expert.

ROHO[®] 